**3M ESPE**

**Clinpro™ Glycine Prophy Powder**

**Prophy Powder for Gentle Sub and Supragingival Cleaning**

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3M, ESPE and Clinpro are trademarks of 3M or 3M Deutschland GmbH. © 2019, 3M. All rights reserved.

the manufacturer of the air polishing device or dental chair if you have any questions about the settings.

- ▶ Aim the spray nozzle, e.g., into a damp washbasin and spray for approximately 10 seconds until the powder-water mixture becomes homogeneous.
- ▶ Spread petroleum jelly on the corners of the patient's mouth to avoid drying and cracking.
- ▶ Place a saliva ejector in the corner of the patient's mouth so that the area under the tongue is suctioned clean.
- ▶ To suction the water-powder mixture deflected by the teeth, use the large suction canula.

**Treating the patient**

- ▶ **Supragingival treatment or exposed root surfaces:** Hold the nozzle near the tooth surface at a maximum distance of 4 mm and an angle of 30° to 90° to the tooth surface; spray each side of the tooth for a maximum of 5 sec., constantly moving the nozzle in a circular motion.
- ▶ **Subgingival treatment:** Hold the nozzle near the gingival margin at a maximum distance of 4 mm and an angle of 30° to 60° to the tooth surface. Spray directly on the tooth along the gingival margin in a smooth, steady motion to have the powder clean subgingivally. The smaller the angle, the deeper the powder stream will penetrate the sulcus and the less deflected the water-powder stream will be.
- ▶ Avoid aiming the spray continuously at one spot. Spray along the surface to ensure an even cleaning effect.
- ▶ Keep the suction canula near the tooth being sprayed during treatment.

**Prepare the Device for the Next Treatment**

- ▶ Empty the powder chamber of the device at the end of the day and remove the powder residue completely with dry air. Subsequently re-close the powder chamber and start the device for approx. 5 sec to remove all powder residue from the device and its lines.
- ▶ If you use a variety of powders, empty the powder chamber of the device and blow dry air through it to remove any powder residue before every powder change. Do not allow the prophylactic powder to come into contact with moisture; moist powder can cause blockage of the nozzle.
- ▶ Clean the device according to its Operating Instructions.

**Storage and Stability**

Always store Clinpro Glycine Prophy Powder in a closed, dry, clean container. Store the product at 15–25 °C/59–77 °F. Do not use after the expiration date.

**Customer Information**

No person is authorized to provide any information that deviates from the information provided in this instruction sheet.

**Warranty**

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

**Limitation of Liability**

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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**ENGLISH**

**Product Description**

Clinpro™ Glycine Prophy Powder is a prophylactic powder for gentle and professional sub- and supragingival cleaning of teeth, including the removal of plaque-biofilm and stains, using commercially available air polishing devices. When applied subgingivally, it may be used as an adjunct to support periodontitis and perimplantitis therapy.

This symbol on the bottle label means: Shake bottle before use.

- ▶ These Instructions for Use should not be discarded for the duration of product use. The product may only be used when the product labeling is clearly readable.

**Indications**

- Professional cleaning of teeth: Removal of subgingival and supragingival plaque-biofilm and stains.
- Can also be used in the presence of fixed orthodontic devices (brackets), restorative and prosthetic materials, and implants.
- For maintenance in periodontitis therapy after completion of initial treatment – when using standard devices – for gingival pockets up to 5 mm in depth.
- For maintenance in perimplantitis therapy after completion of initial treatment – when using standard devices – for gingival pockets up to 5 mm in depth.

**Precautionary Measures**

- Calculus and tartar cannot be removed with Clinpro Glycine Prophy Powder.
- Caution should be exercised when using Clinpro Glycine Prophy Powder for treatment of patients suffering from chronic bronchitis or asthma because the fine spray of air and powder can cause respiratory problems.
- When using an air polishing device, both operator and patient must wear protective eyewear during use. Powder accidentally sprayed into the eyes can cause severe eye injury. The fine powder dust can cause irritation if it gets under contact lenses. Patients wearing eyeglasses should wear protective eyewear over their eyeglasses to avoid getting them soiled during treatment.
- You may wear a mask as protection from dust particles (e.g., when filling the device).
- Do not aim the nozzle directly at the gum tissue during treatment. Proper positioning of the nozzle is described under "Treating the Patient". As with all prophy powders, there is a risk of provoking a subcutaneous emphysema when compressed air is used intracavally.
- Please pay attention to possible restrictions when treating high-risk patients (general weakened immunity, endocarditis) because bacteremia may occur when treating deep periodontal pockets.
- Always use a suction device during treatment.
- Never use the air polishing device without water as this makes it more difficult to suction the air-powder mixture, causing injuries in the area to be treated.

3M MSDSs can be obtained at [www.3m.com](http://www.3m.com) or from your local subsidiary.

**Use**

Clinpro Glycine Prophy Powder is intended for use in commercially available air polishing devices. Please follow the Operating Instructions to operate the specific device.

**Fill with powder**

- ▶ Completely remove any residue of other cleaning powders by blowing air through the device; never use bicarbonate or carbonate powder for subgingival cleaning! – Do not use water or damp air to clean the device because damp powder can cause blockage of the nozzle.
- ▶ Shake the bottle before filling powder into the powder chamber of the device.
- ▶ Snap the spout of the bottle upward.
- ▶ Add prophylactic powder into the powder chamber of the device until it reaches the maximum level (see marking on the device). Fill slowly in order to avoid dust.

**Settings on air polishing device**

The cleaning efficiency of the prophylactic powder is dependent on a number of factors, including the air polishing device being used and the settings for air pressure and water flow.

- ▶ Follow the Operating Instructions of the device and test the settings to determine what is optimal for your use before beginning treatment. Contact