AIR-FLOW® SOFT

PROPHYLAXIS POWDER

Before using this product, please carefully read and follow the recommendations of the instruction manuals originally delivered with your device and system. Please pay special attention to the safety precautions.

Product Description

The AIR-FLOW® Soft (prophylaxis powder) is used solely for supragingival treatments.

AIR-FLOW® Master and AIR-FLOW® Master Piezon users: this powder has to be used only with the AIR-FLOW® or the AIR-FLOW® CLASSIC+SOFT powder chamber.

This powder is for use in dentistry with the EMS air polishing devices.

Applications

This powder is particularly suited for repeated treatments. It is recommended for the following cleaning procedures:

- Effective plaque and stain removal for orthodontic patients,
- Plaque removal prior to fluoride treatment,
- Plaque and stain removal prior to whitening procedure,
- Plaque removal for placement of sealants,
- Surface preparation prior to bonding/cementation of inlays, onlays, crowns and veneers,
- Surface preparation prior to placing composite restorations,
- Cleaning prior to bonding orthodontic brackets,
- Clean implant fixture prior to loading,
- Stain removal for shade determination.

Contra-indications

⚠ Patients suffering from upper respiratory tract infections, from chronic bronchitis or asthma must not, under any circumstances, be treated with an air polishing device. The jet of air and powder could cause respiratory difficulties.

⚠ Predisposed persons may be sensitive to glycocol. If allergic reactions are observed, stop using this product and completely remove the powder from mouth.

⚠ Do not direct the jet of powder toward fillings, crown or bridgework as this could damage these restorations.

⚠ Do not use this powder subgingivally. For this application use only the powder recommended by EMS for subgingival application.

Recommendations for the treatment

⚠ Patient and user have to wear eye protection.

⚠ User has to wear a mask and gloves.

⚠ Use the high-speed evacuator of your dental unit.

⚠ Apply Vaseline® on the patient's lips.

⚠ Wash hands after use.

⚠ Wash patient face after use (e.g. with wet wipes).

General information

• Do not exceed the powder filling limit as indicated on the powder chamber.

• This powder is sugar free.

Storage and shelf-life

• The powder should not be stored near heat sources.

• Storage condition up to 25°C and 10% to 90% relative humidity.

• Transport conditions -29°C to 38°C and 10% to 90% relative humidity.

• Do not use after expiration date.

Also available:

• AIR-FLOW® CLASSIC available in Lemon, Mint, Cassis, Cherry, Tropical and Neutral (for supragingival treatments).

• AIR-FLOW® CLASSIC COMFORT (for supragingival treatments).

• AIR-FLOW® PERIO (salt free for subgingival application).

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